# Bluephase® EasyCure

## Start-up

- 1 ✓ Unpack handpiece  
✓ Clean the handpiece and light guide

- 2 ✓ Connect the power cord with the power supply

- 3 ✓ Slide battery straight into the handpiece until you hear and feel it click into place.


- 4 ✓ Place handpiece in the corresponding rest in the charging base

**i** If the battery is fully discharged, the charging time is approx. 2 hours

**i** Disposable protective sleeves are recommended for each patient. Attach anti-glare cone or anti-glare shield to the light guide.

## Operation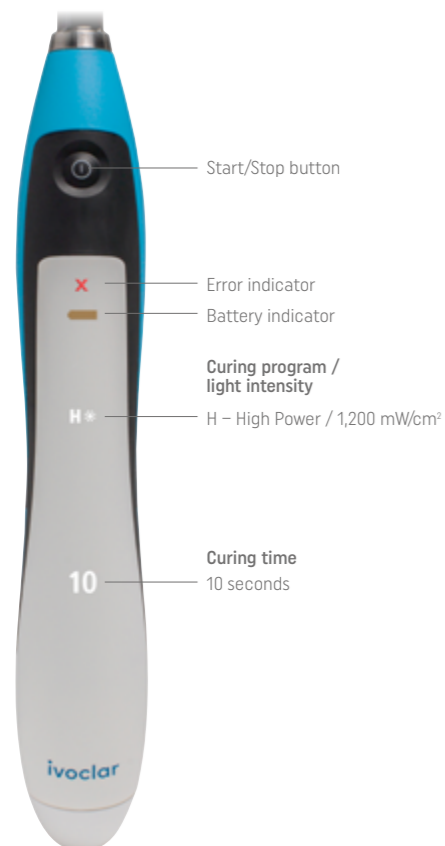

## What if ...?

If any of these errors persist after trying their corresponding rectification, please contact your dealer or your local Service Centre.

Indicator	Causes	Error rectification
Red "X" is flashing	The device is overheated.	Allow the device to cool down and try again after a certain time. If the error persists, please contact your dealer or your local Service Centre.
Red "x" lights permanently	Electronic component of the handpiece is defective.	Remove and reinsert the battery. If the error persists, please contact your dealer or your local Service Centre.
Red "x" lights permanently and the battery symbol is flashing	Battery empty	Place the device in the charging base and charge it.
	Battery contacts dirty	Remove battery and clean the battery contacts.
The charging base is not illuminated during charging	- Power cord not connected - Battery fully charged	Check if the charging base is connected to the power supply by means of the power cord.

## Cleaning

- ✓ Clean the handpiece with the light guide attached using a commercially available, aldehyde-free disinfectant solution.
- ✓ After cleaning, remove the light guide and allow it, as well as the handpiece, to dry for one minute.

\* Damage to the Bluephase curing light caused by the use of cleaning agents, including discolouration and corrosion, is not covered by the Bluephase curing light warranty.

**IN THE VIDEO YOU WILL FIND FURTHER TIPS FOR PRODUCT CARE**

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